

ANORECTAL PROBE

This device is intended for use with
Biofeedback Monitors Model 5 and Model 10

FOR SINGLE PATIENT USE ONLY

CONTENTS:

1 Silicone Anorectal
Probe (adult)


1 Patient Log Book
for Incontinence

1 Patient Log Book
for Constipation

Syringe Not Included

- ① Inflation Valve for Stool
Simulation Balloon
- ② Connector to Monitor
- ③ Stool Simulation Balloon
- ④ Sensor
- ⑤ Black Marker Ring

LOT

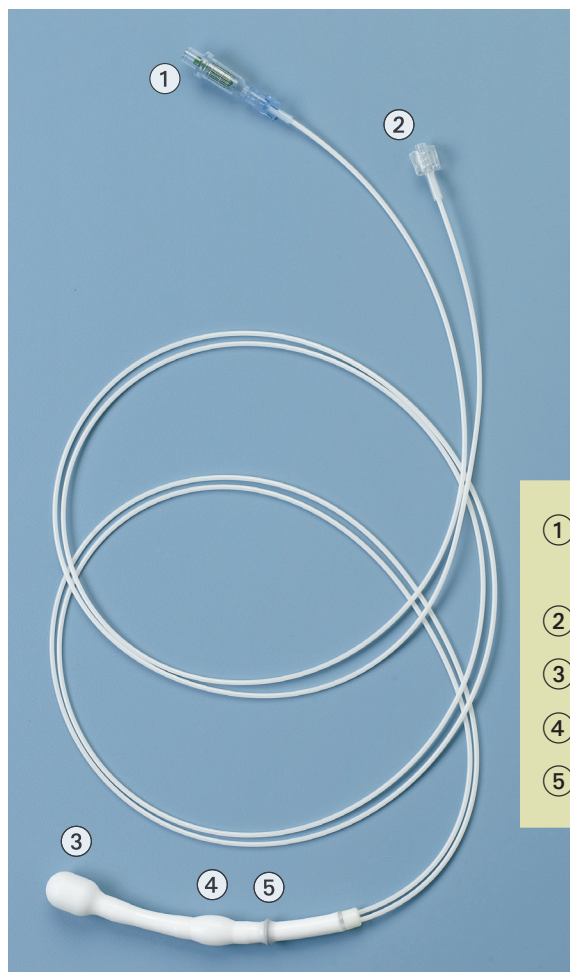

ANORECTAL PROBE

This device is intended for use with Biofeedback Monitors, Model 5 and Model 10.

FOR SINGLE PATIENT USE ONLY

- Medical grade silicone anorectal probe is anatomically designed for patient comfort and biocompatibility.
- Anatomical design responds to appropriate muscle activity.
- Single patient use avoids cross contamination.
- Replacement probes are available.

BEFORE USE: (Test the Probe)

NOTE: Inspect contents of package for damage.
Do not use if damaged.

If using the Distal Stool Simulation Balloon,
please refer to diagram on cover:

- 1) Obtain a luer slip syringe (30-35cc recommended). Draw 20cc of air into the syringe.
- 2) Connect the syringe to the Inflation Valve ① so that any injected air will flow to the Stool Simulation Balloon ③.
- 3) Gently inflate the Stool Simulation Balloon ③ with approximately 20cc of air. Remove the syringe from the Inflation Valve ①.
- 4) Keep the Stool Simulation Balloon ③ inflated for one (1) full minute to ensure there are no leaks in the Stool Simulation Balloon ③. If the injected air has not leaked out, the Stool Simulation Balloon ③ is functioning normally.
- 5) To deflate the Stool Simulation Balloon ③, insert the syringe without the plunger into the Inflation Valve ①.
- 6) Refer to the Biofeedback Monitor Instruction Manuals, 5 and 10 for use of the probe with the monitor.

CARE / MAINTENANCE

This device is sold non-sterile. Before use, rinse the probe with warm water and gently wash with mild soap or soap solution. Be very careful not to come into contact with anything sharp or abrasive. This could damage the balloons.

NOTE: The use of a condom to cover the probe while in use is highly recommended.

The probe may be disinfected by immersion in cold disinfecting solution (i.e. Cidex ®). Follow the disinfectant manufacturer's instructions carefully.

When the process is complete, inflate the Stool Simulation Balloon ③ with 30cc of air and thoroughly rinse the probe (patient contact portion) to remove any residuals. With a soft cloth, gently and thoroughly wipe the probe.

Deflate the Stool Simulation Balloon ③ and allow the probe to air dry in a horizontal position with the tubes forming a loose loop. Store in a plastic zipper bag.

CAUTION:

DO NOT AUTOCLAVE

DO NOT BOIL

DO NOT USE ABRASIVE CLEANERS OR BRUSH

DO NOT USE CLEANING SOLUTIONS THAT CONTAIN AROMATIC, CHLORINATED, KETONE, ETHER OR ESTER BASED SOLVENTS.

* REPLACEMENT PROBES MAY BE PURCHASED SEPARATELY.
REORDER NUMBER: 18-1000



biosearch®
MEDICAL PRODUCTS INC.

35 Industrial Parkway
Branchburg, NJ 08876-1276 U.S.A.
Phone: 908-722-5000
Fax: 908-722-5024
<http://www.biosearch.com>



Emerge Europe
Molenstraat 15
2513 BH The Hague
Netherlands
Phone: +31 (0)70 345 8570
Fax: +31 (0)70 346 7299



© 2011 Biosearch Medical Products, Inc.
All rights reserved

3945 Rev.07/11